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**Dental and Medical Products**  
for home care, practice, laboratory and hospital

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**Since 1950** the family-owned company Schür has been using the unique material melamine to produce innovative products for the food and medical sector under the brand names FESTOPAS® and NOVOTRAY®. The aim of the company's endeavours still is and always has been to focus on just one material and to become the foremost specialist and quality leader in the market. Founded by Anneliese and Wilhelm Schür, the family business is now being managed by their daughters, Christine Schür and Sabine Beer.

## Creative Symbiosis of Human and Technology

The manufacture of FESTOPAS® and NOVOTRAY® products made from pure melamine is a combination of work done by hand and high-end technology. Hydraulic presses with a compression capacity of up to 250 tons mould the products out of the base material melamine granules. Then each blank is inspected, deburred and polished by the competent hands of our employees to achieve the final surface finish.

## Quality Guaranteed

We work within a product-oriented quality management system and are monitored and quality-controlled by the government testing institute in Darmstadt. All products are manufactured and finished in Germany at our location Verl in East-Westphalia. The melamine raw materials are sourced exclusively from Italy from manufacturers that employ EU-DIN standard manufacturing processes.

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## FESTOPAS® and NOVOTRAY® Professional Quality

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### Environment and Future

The stability and longevity of our products take into account the notions of sustainability and environmental protection. Pure melamine is a thermosetting material, made from the natural raw materials of coal, lime, cellulose and synthetic resins – but without crude oil.

We take back all used FESTOPAS® and NOVOTRAY® products as long as they are delivered to us in clean condition, so as to recycle them or to find a suitable ecological re-use purpose for the products.

### Performance Characteristics

- ceramic-like look
- highly shiny, non-porous and very hard surface
- lightweight, stackable, low-noise, shatter-proof and shock-proof
- solid-coloured and light-resistant
- dimensionally stable over the long term
- anti-static, fire-proof
- resistant to cleaning agents and disinfectants, alcohol, ether, oils, weak acids and bases.
- heat-resistance: max. transient temperature +120 °C, max. sustained temperature +70 °C
- recommended for steam sterilization in the autoclave up to 120 °C
- recommended for cleansing in the thermo disinfectant when using the recommended cleansers
- can be sterilised with formalin and ethylene oxide, not dry
- dishwasher-safe and antifreezing down to –50 °C (long-term)
- easy to identify using colour coding
- easy to clean (for care and reconditioning instructions, see page 12)

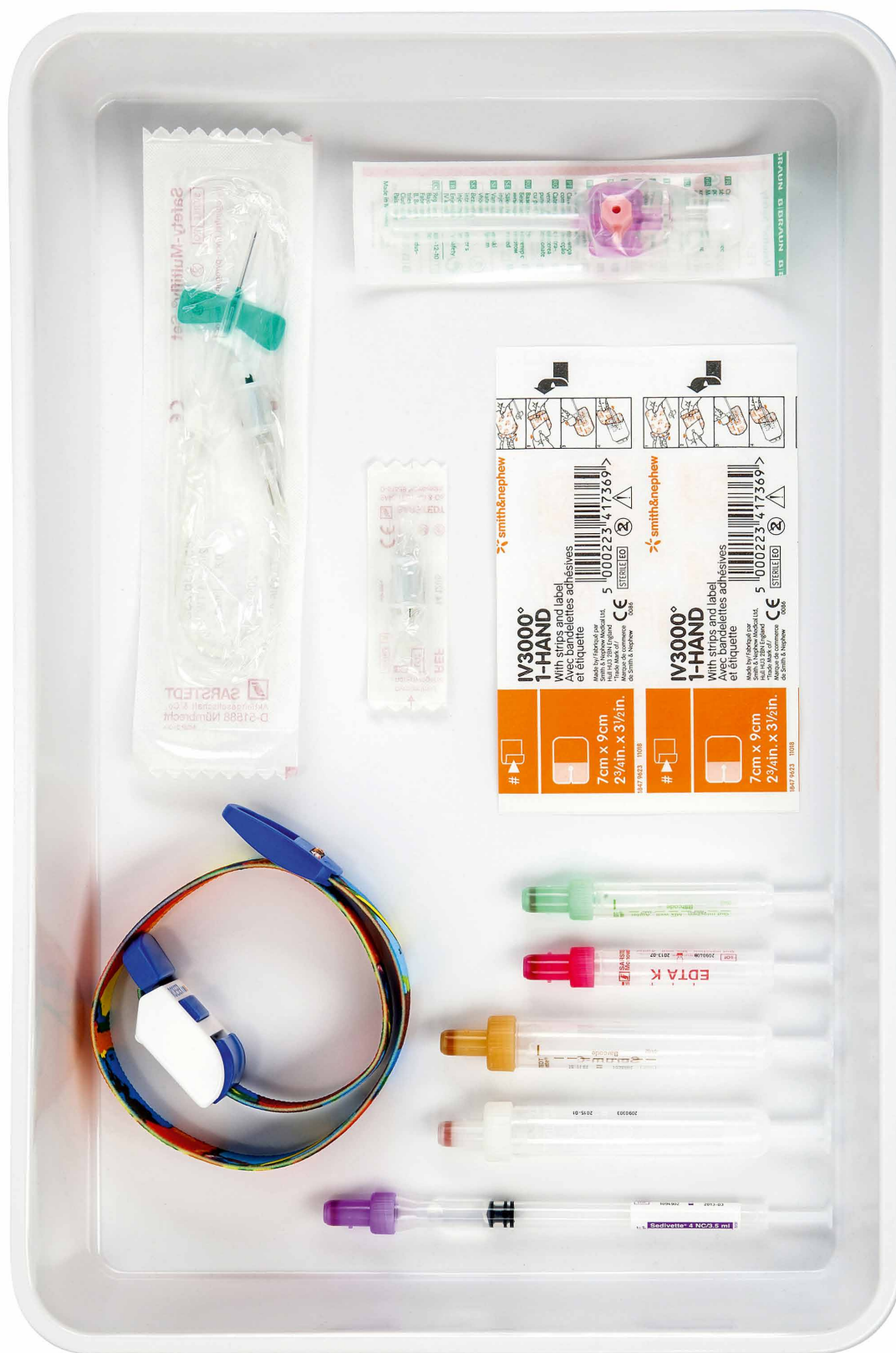
### Colours

Put together your individual surgery equipment. All sizes and shapes are available in many of the colours shown on the left.

### Colour Service

Upon request, we can produce FESTOPAS® and NOVOTRAY® in further colours. Please contact your retailer, who will gladly request the relevant conditions and order quantities from us.

FESTOPAS® und NOVOTRAY® Profi-Qualität  
 FESTOPAS® and NOVOTRAY® Professional Quality

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### **FESTOPAS® and NOVOTRAY® – The extra Dose of Hygiene**

Style and substance don't have to be opposites. Visible and tangible proof: FESTOPAS® and NOVOTRAY® made from pure melamine, the solid surface material, fascinatingly beautiful and simply practical.

Melamine combines the benefits of natural materials with the shock-resistance and non-porous qualities of acrylic, although it is insensitive to weak acids and alkalis, cleaning agents and UV light. And extremely hygienic.

A further advantage is that plastics like melamine do not oxidise! In contrast to aluminium and steel, there are no corrosion problems, no tarnishing of the surfaces, no ugly discolourations. The materials are 100 % solid-coloured, nothing can peel, come away or swell.

The care and reconditioning procedures are very simple. You can find our manufacturer information for reconditioning in accordance with **DIN EN ISO 17864:2004** enclosed on page 12 and for download on our website **www.schuer.biz**.

# NOVOTRAY® Normtrays / Dentalcassetten und Zubehör

## NOVOTRAY® Norm Trays / Dental Cassettes and Accessories

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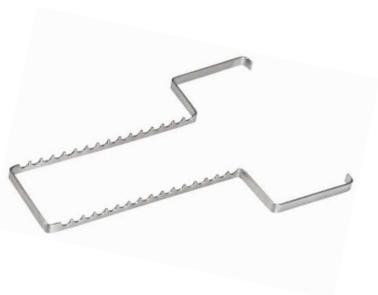
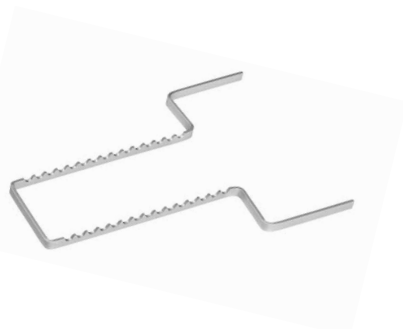
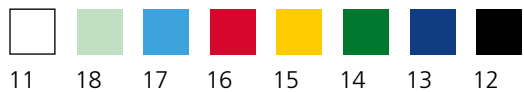
## FESTOPAS® Instrument Bowls, Covering and Stacking

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### Possibilities of Covering and Stacking the Trays

Almost all bowls can be covered and stacked to save storage space by selecting the corresponding FESTOPAS® tray of the same length x width dimension, for example the bowls of Art.-No. 68/0 and 68/1 can be covered with trays of Art.-No. 77/0.

# Herstellerinformation Manufacturer's Information

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## For the reconditioning of Festopas® and Novotray® Bowls and Trays in accordance with DIN EN ISO 17864:2004

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### PRODUCTS

FESTOPAS® bowls and trays and NOVOTRAY® norm trays made from melamine

### LIMITATIONS ON RECONDITIONING

Can be reconditioned practically without limitations

### WORKPLACE

Remove surface contamination with a disposable paper towel.

### STORAGE AND TRANSPORTATION

No particular requirements are placed on storage and transportation.  
We recommend reconditioning the products as soon as possible following use.

### CLEANING PREPARATIONS

None

### MANUAL CLEANING AND DISINFECTION

#### Necessary equipment

Cleaning agents e.g. mild detergent or disinfectant sprays or disinfectant cloth, brush, running water

#### Procedure

1. Clean surface contamination from the product under running warm water and, if necessary, with the aid of a brush.
2. Apply the chosen cleaning agent to all surfaces. Every area must be reached.
3. Then, thoroughly rinse the product under running water, before drying the product with a clean cloth.
4. Apply the disinfectant to the product in accordance with the manufacturer's instructions, give it time to act (note the minimum duration of action) and, if necessary, wipe with a disposable cloth.

### MACHINE-BASED CLEANING AND DISINFECTION

#### Necessary equipment

- Cleaning /disinfecting device
- Cleaning agent (according to the device manufacturer's instruction or see recommendation for thermo disinfectors), brush

#### Recommended cleaning agent for the thermo disinfectant

- Dr. Weigert MediClean Dental
- If required, also small quantities of Neodisher N Dental to neutralise and Neodisher MediKlar Dental as a rinsing agent.

#### Procedure

1. Load the product into the device
2. Select standard cycle, wash for at least 5 minutes and rinse for 3 minutes (when selecting the programme, do not exceed a temperature of +120 °C or only do so temporarily).
3. When unloading the device, inspect the product for visible contamination. If necessary, clean manually with a brush and repeat machine-based cleaning.
4. To disinfect, a rinsing process can then take place for 10 minutes at +80 °C.

#### Maintenance / Inspection and performance test

After each reconditioning process. In the event of damage which affects the usability of the product, the product should be disposed of.

### MANUFACTURER'S INFORMATION

The manufacturer has validated that the above instructions for the preparation of a medical product are SUITABLE for the reconditioning of this product. The person reconditioning the product bears responsibility for ensuring that the reconditioning performed with the available equipment, materials and personnel achieves the required results. Similarly, any deviation should be carefully evaluated for its efficacy and possible consequences.

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Schür Kunststofftechnik GmbH & Co. KG
**Schür Kunststofftechnik GmbH & Co KG**